

WALACH

PAPETTE[®] Cervical Cell Collector (STERILE)

REF 908000-1



Read all safety information and instructions listed below before using this product!

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

INTENDED USE

- For Cervical Cytology utilizing the Conventional Pap Smear.
- The *PAPETTE* is the most gentle, yet effective means of simultaneously taking both ectocervical and endocervical cell samples.

INSTRUCTIONS

1. Contact the cervix with the *PAPETTE* and insert the central bristles into the cervical canal deep enough to allow the shorter bristles to fully contact the ectocervix.
2. Push gently, and rotate the *PAPETTE* in either direction.
3. Remove the *PAPETTE* and draw each side once across the slide (like a paint brush), thus transferring both ectocervical and endocervical cell samples in one stroke.
4. Dispose of *PAPETTE* in accordance with local Medical Hazardous Waste practices.

Note: It is advisable to inform your pathologist or lab that the Wallach *PAPETTE* is being used to obtain samples.

PAPETTE[®] U.S. Patent #4,700,713

Medguard
Professional Healthcare Supplies
Tel: +353 (01) 835 2411 Email: service@medguard.ie
Fax: +353 (01) 835 2461 Web: www.medguard.ie