



Ergoscan Ergoline- ABPM


HOW TO PRINT PDF

Medguard
Professional Healthcare Supplies
Tel: +353 (01) 835 2411 Email: service@medguard.ie
Fax: +353 (01) 969 5009 Web: www.medguard.ie


Contents

Contents.....	2
1. Open up software package and click 'Analyse'	3
2. Select patient from 'Patient Search' Record.....	4
3. Click 'Select' in Patient data	5
4. Choose required report and select 'Print'	6
5. Select PDF printer from 'Printer' options	7
6. Preview your document and click 'Print' icon at bottom of the page.....	8
7. Choose location to store your file in 'Save In' options and then click 'Save'	9
8. Patient file shall now be saved to your chosen location	9


1. Open up software package and click 'Analyse'

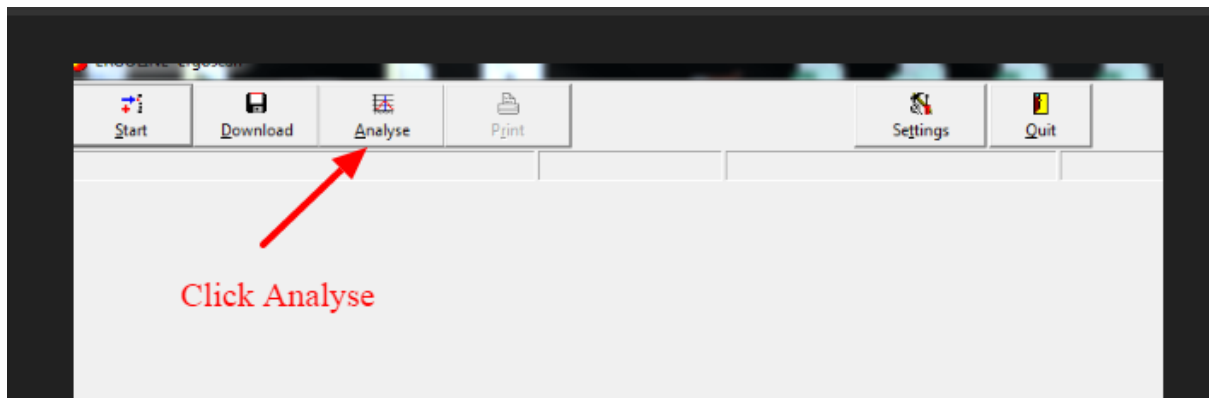

2. Select patient from 'Patient Search' Record

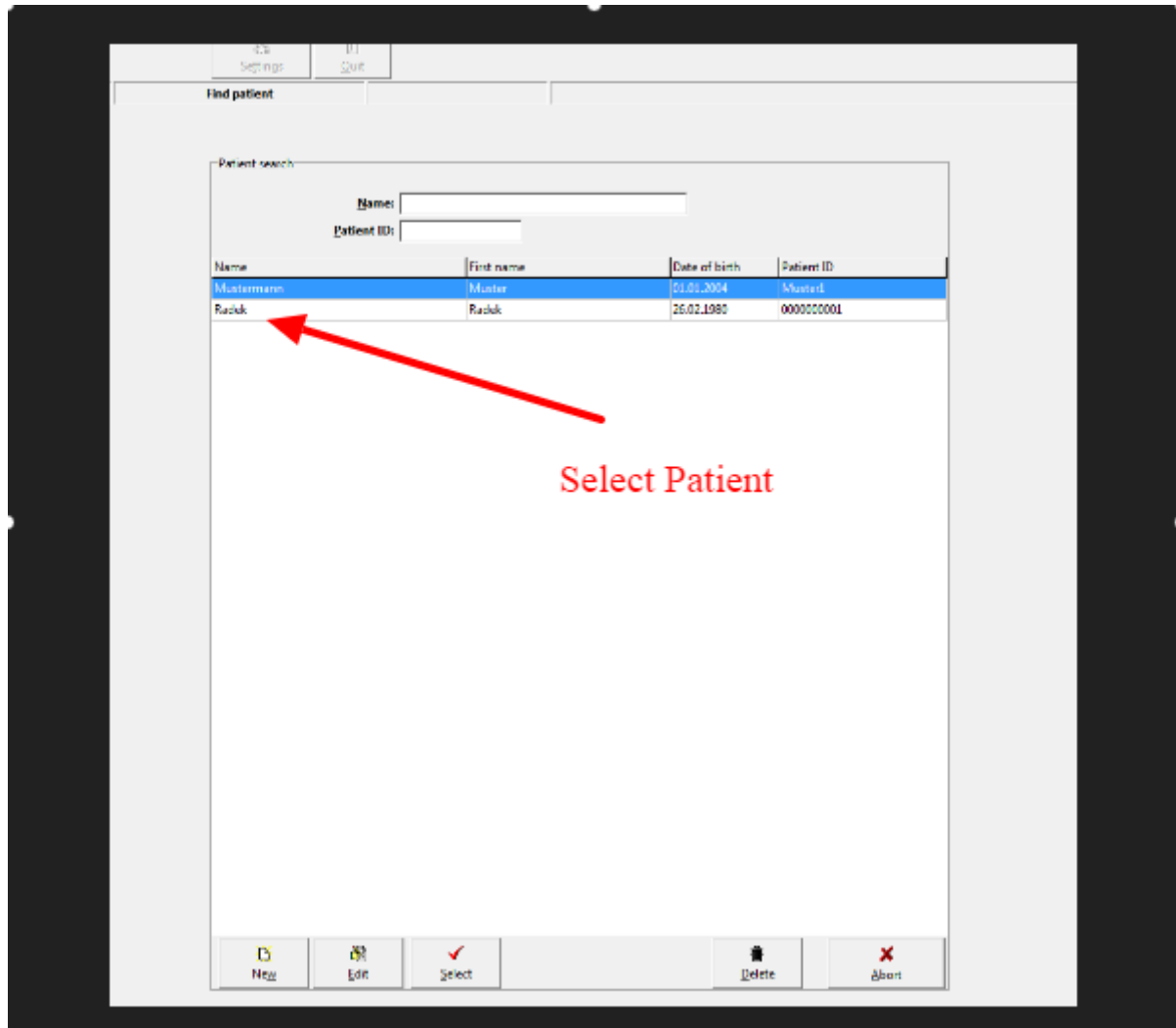


3. Click 'Select' in Patient data

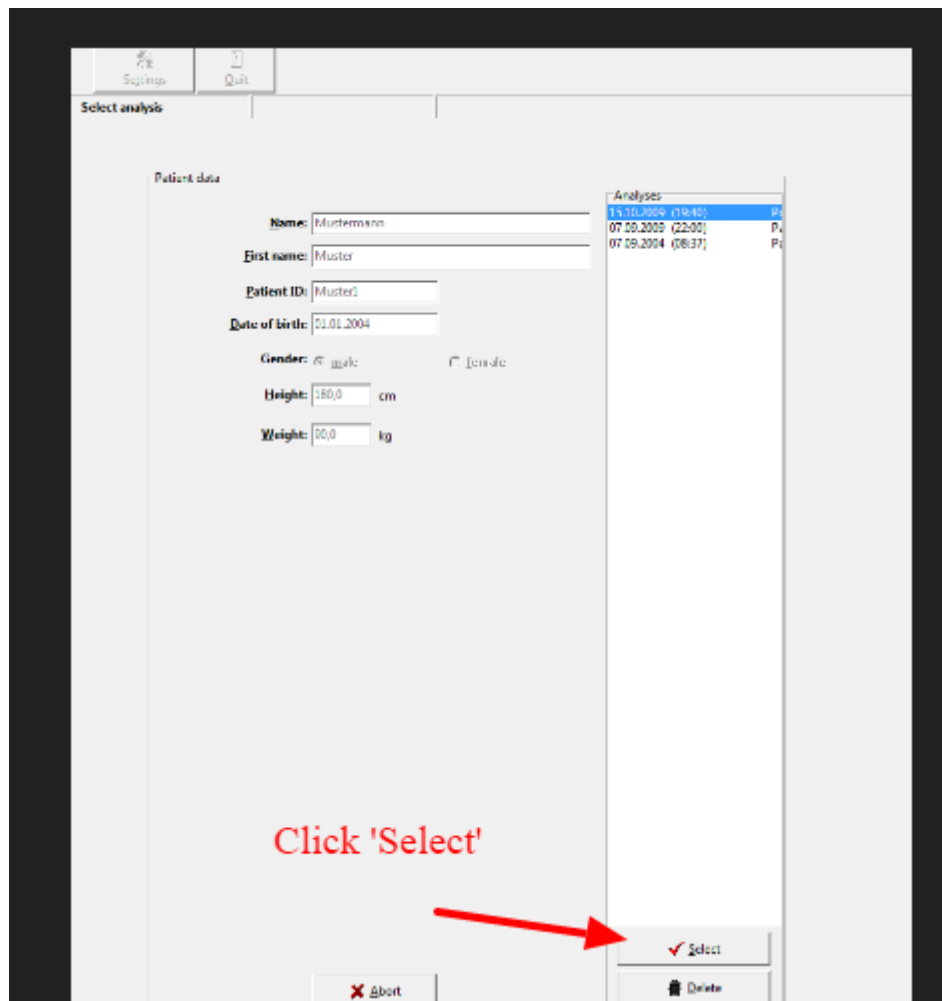

4. Choose required report and select 'Print'



5. Select PDF printer from 'Printer' options

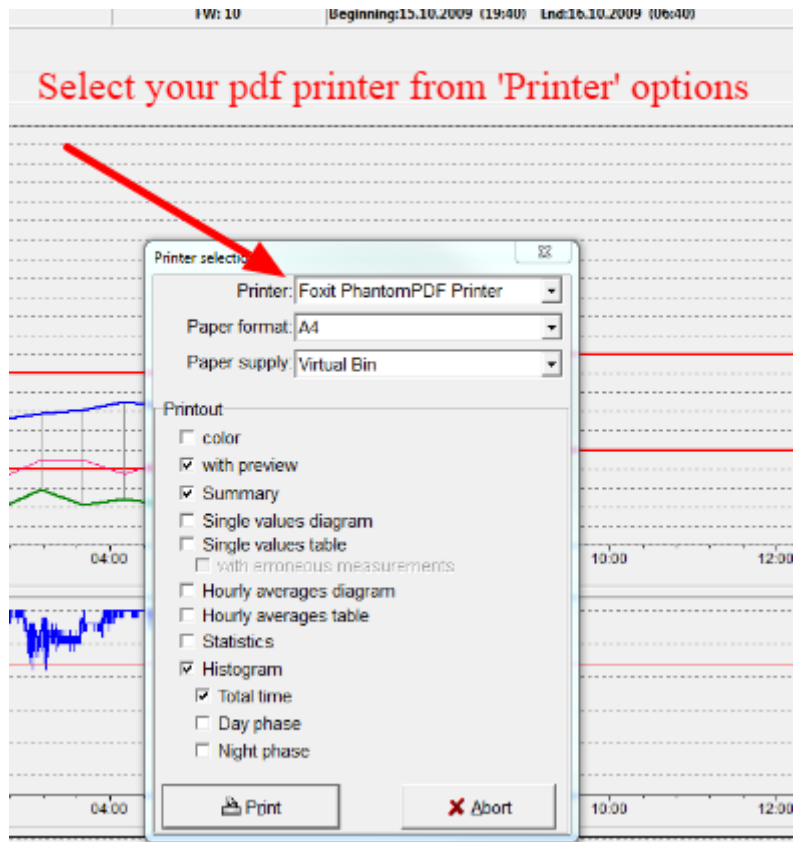

6. Preview your document and click 'Print' icon at bottom of the page

Print Preview

Dr Basim Alwar
Medguard
Unit 205
Ashboum Business Centre
Tel: 0851 686760
Fax: 00000000000

Patient name: Mustarman Alwaleh
ID: Mustar
Date of birth: 01.01.2004
Height: 180.0 cm
Weight: 93.0 kg
Measurement: 15.10.2009 (19:43)


Summary				
Measurement from 15.10.2009 (19:43) to 16.10.2009 (08:42) Duration: 11:59 Hours				
	Totaltime	Dayphase	Nightphase	Day/night difference
Blood pressure (mmHg)	124 / 70	137 / 81	118 / 74	-13.0% / -21.0%
Heart rate (beats/min)	88	77	66	-11.7%
SpO2 (%)	98	100	93	-3.0%
Number of saturations (SpO2 > 92%)	1			
Duration of longest saturation (SpO2 > 92%)	38 sec			
Number of measurements with	25	0	17	
Limit values		140 / 90	130 / 80	
Systolic > limit	6 (24.0%)	3 (37.5%)	3 (17.6%)	
Diastolic > limit	3 (8.0%)	2 (25.0%)	0 (0.0%)	

Diagram of the single val.

Click 'Print'

Findings

7. Choose location to store your file in 'Save In' options and then click 'Save'

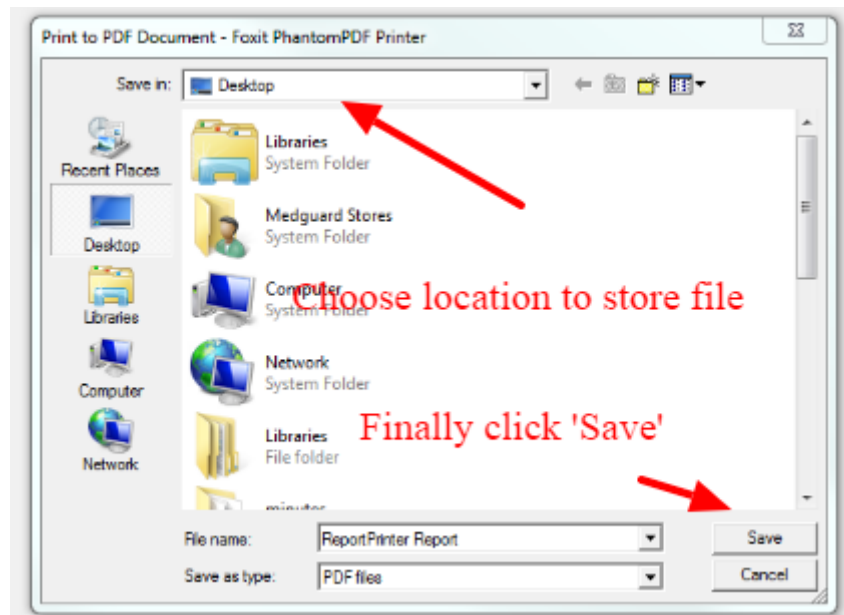

8. Patient file shall now be saved to your chosen location