

# GENERAL ENVIRONMENTAL HYGIENE

## Actichlor™

Chlorine Disinfectant Range

Comprehensive range of chlorine disinfectant products for all aspects of surface and environmental disinfection.

Actichlor also provides:

- ▲ Full range of formats and concentrations
- ▲ Broad spectrum efficacy
- ▲ Simple to use



# Actichlor™

Chlorine Disinfectant Range

## Why Use Actichlor™ Plus?

### EXCELLENT CLEANING AND DISINFECTION PERFORMANCE IN ONE EASY STEP

The Actichlor Plus tablet combines a chlorine compatible detergent with NaDCC in a single tablet format.

## Disinfect with Actichlor™

A comprehensive range of chlorine disinfectant products for all aspects of surface and environmental disinfection:

- ▲ Full spectrum of efficacy
- ▲ Wide range of tablet sizes available

### MICROBIOLOGICAL PROPERTIES

Disinfectant solutions made with Actichlor are effective against all bacteria, spores, yeasts, moulds and viruses including Hepatitis B and HIV.

Tested to EN 1276, EN 1650 and EN 13704.

### PRODUCT SPECIFICATIONS

**Actichlor Disinfectant Tablets** - Effervescent NaDCC disinfectant tablets.

**Actichlor Granules** - Absorbent NaDCC granules for the disinfection of blood spills.

**Actichlor Plus Tablets** - Effervescent NaDCC tablets with detergent.

**Actichlor Spill Kit** - 10 clean up packs, with dilution bottle, disinfection granules and disinfection tablets for blood spills.

### TRAINING AND SUPPORT

To aid product usage, a dilution bottle is available to assist accurate dilution of Actichlor and Actichlor Plus.

The range is also supported by usage posters and a training DVD, available from your Healthcare Manager.

PROVEN.  
EFFECTIVE.  
EFFICIENT.

### APPLICATION AREA

For disinfection of blood spills, general clinical environment, sinks and drains, babies' bottles and catering equipment.

### INSTRUCTIONS FOR USE

To achieve a solution of 1000ppm, dissolve 1 x 1.7g tablet in 1 litre of water; or for a solution of 10,000ppm, 10 x 1.7g tablets in 1 litre of water. Please see dilution tables for additional dilutions.

### INGREDIENTS

Each Actichlor tablet contains Sodium Dichloroisocyanurate (NaDCC, troclosene sodium) in an inert effervescent base.

Actichlor Spill Kit - 10 clean up packs, with dilution bottle and disinfection tablets for blood spills.

### PACKAGING AND ORDERING INFORMATION

PRODUCT	OUTER SIZE	ORDER CODE
Actichlor Plus 1.7g Disinfectant Tablets	1.7g x 150	3029060
Actichlor Foil Disinfectant Tablets	0.5g x 80	3050740
Actichlor 0.5g Disinfectant Tablets	0.5g x 250	3049690
Actichlor 0.5g Disinfectant Tablets	0.5g x 600	3049680
Actichlor 1.7g Disinfectant Tablets	1.7g x 30	3049170
Actichlor 1.7g Disinfectant Tablets	1.7g x 200	3037640
Actichlor 2.5g Disinfectant Tablets	2.5g x 100	3049670
Actichlor 5g Disinfectant Tablets	5g x 50	3050650
Actichlor 500g Granules	500g	3037680
Actichlor Spill Kit	1	3030520
Actichlor Dilution Bottle	1	408432



Actichlor Plus 1.7g Disinfection Tablets



Actichlor 0.5g Disinfection Tablets



Actichlor 1.7g Disinfection Tablets



Actichlor Granules



Actichlor Spill Kit

Medguard Professional Healthcare

Tel: +353 (0) 1 835 2411

Email: sales@medguard.ie

Fax: 01 969 5009

Website: www.medguard.ie

Unit 6/7, Block 4, Ashbourne Business Park, Ashbourne, Co. Meath, A84 VK52

**ECOLAB®**