

**QUICK, SANITARY
CLEAN-UP.**



Medline Emesis Bag

Wherever emesis occurs, the emesis bag has it handled.

- Emergency department
- Post-operative
- Ob-gyn
- Wards
- Ambulance
- Home healthcare
- Paediatrics
- Oncology
- Cancer treatment centres
- ICU



Blue emesis bag EMB01




Clear emesis bag EMB02

Item Number	Description	Packaging
EMB01*	Emesis bag - blue	144/cs
EMB02*	Emesis bag - clear	144/cs
DISP01	Dispenser for emesis bags, wall-mounted, holds 24 pieces	6/cs

Using the emesis bag is as easy as 1-2-3.


1

Prepare for use. Hold the comfort ring and pull the plastic bag down gently and away from the ring. This unfolds the bag.

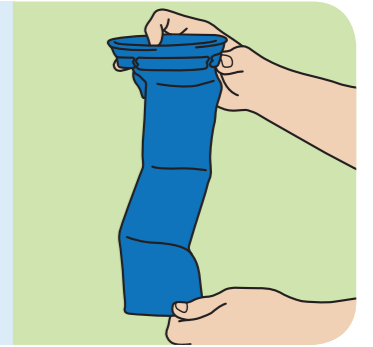

2

Use the emesis bag. Cover the mouth with the comfort ring during regurgitation, directing the vomitus into the bag until finished.

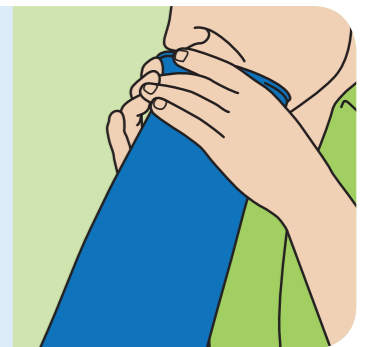

3

Secure the contents. Grip the bag 8 to 10 centimeters below the comfort ring and twist the ring. Tuck the tightly twisted bag into one of the four notches around the ring to hold it closed. Dispose of it in a sanitary manner.



Dispenser DISP01

Discover Medline Emesis Bags.

Improved patient experience

- Less splashback and mess than with basins
- More privacy and discreetness for patient dignity
- Enhanced independence and self-help

More sanitary

- Contained and covered vomitus
- Isolated contaminants and pathogens
- Less cross-contamination on floors and surfaces

Quicker clean-up

- Easier to clean up the patient after sickness
- Fewer linen changes and less floor mopping
- Disposal in regular or red bag trash



*This product is a class I non-sterile medical device intended to be used by healthcare professionals. Before use, consult instructions for use and precautions on the corresponding labelling.

Medline International France SAS,
5 rue Charles Lindbergh
44110 Châteaubriant, France



Available now!

For more information, please contact your Medline account manager or visit our website: www.medline.eu/uk

Medguard Professional Healthcare

Tel: +353 (0) 1 835 2411

Email: sales@medguard.ie

Fax: 01 969 5009

Website: www.medguard.ie

Unit 6/7, Block 4, Ashbourne Business Park, Ashbourne, Co. Meath, A84 VK52