

# URINE SAMPLE KITS

## USER GUIDES

# MEDGUARD®

Professional Healthcare Supplies

### URINE SAMPLE KIT WITH FUNNEL (UN252)



01 Wash your hands thoroughly before you proceed.



02 Peel open the pack and remove sterile container.



03 a) Pass a small quantity of urine into the toilet and without interrupting the flow of urine collect approximately 10 ml in the container.  
b) Finish passing urine into toilet.



04 Detach and discard plastic funnel from the container as per Trust guidelines. Fit cap provided securely onto container.



05 Wash your hands, write your name on label provided.

### URINE SAMPLE KIT WITH CLEANSING WIPE (UN257)



01 Wash your hands thoroughly before you proceed.



02 Peel open the pack and clean genital area with the cleansing wipe provided.



03 a) Pass a small quantity of urine into the toilet and without interrupting the flow, collect approx 10 ml in the container.  
b) Finish passing urine into toilet.



04 Detach and discard plastic funnel from the container as per Trust guidelines. Fit the cap provided securely onto container.



05 Wash your hands, write your name on label provided.

### URINE SAMPLE KIT WITH BORIC ACID (UN258)



01 Wash your hands thoroughly before you proceed.



02 Peel open the pack and remove plastic stopper. Please note the funnel and white powder must remain in the container.



03 a) Pass a small quantity of urine into the toilet and without interrupting the flow, collect approx 10 ml in the container.  
b) Finish passing urine into toilet.



04 Detach and discard plastic funnel from the container as per Trust guidelines. Fit the red cap provided securely onto container.



05 Wash your hands, write your name on label provided.

The sequences shown here should be used as guidance. Please refer to local hospital guidelines for further information.

## FOR MORE INFORMATION

Medguard Professional Healthcare

Tel: +353 (0) 1 835 2411

Email: [sales@medguard.ie](mailto:sales@medguard.ie)

Fax: 01 969 5009

Website: [www.medguard.ie](http://www.medguard.ie)

Unit 6/7, Block 4, Ashbourne Business Park, Ashbourne, Co. Meath, A84 VK52