


SPINAL BOARD

# MEDGUARD®

Professional Healthcare Supplies


USE AND MAINTENANCE BOOK

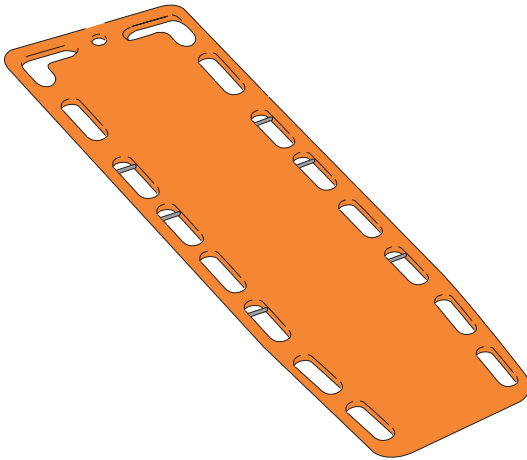

**ATTENTION:** The operators must carefully read and completely understand the present manual before using the product.



CE

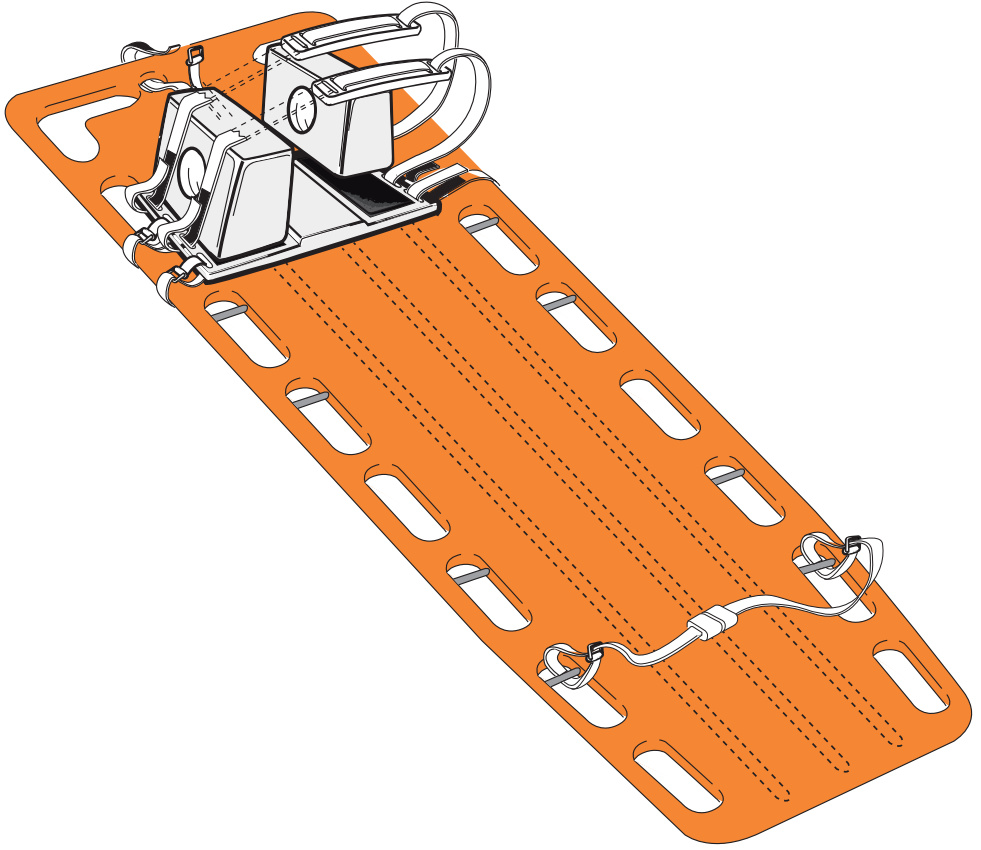



This product is made of high density PE material by one-step molding. There are two carbon fibre reinforced plastic stays which makes board strong.

The spinal board is used for transferring spinal injury patient, also used as normal stretcher.

**Dimension:** 72" X 18" X 2" (183 cm X 46 cm X 5 cm)

**Weight:** 7.0 kgs

**Loading capacity:** 200 kgs



**Warning!**

*The product should be practiced by trained persons.*

*The patient should be moved and carried by trained persons, or damage may occur.*

### OPERATION

1. Place the product on flat ground.
2. Move the patient on the product in the right way.
3. Unlock the fixed straps, and thread one side of strap through the handle hole of spinal board, and lock the straps one by one.
4. The spinal board has handles on the edge. After be sure the patient is fixed well, the patient can be moved both frontward and backward or rightward and leftward.

### CLEANING

Wash with water and soap or bactericidal liquid

### GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

### Medguard Professional Healthcare

Tel: +353 (0) 1 835 2411

Email: [sales@medguard.ie](mailto:sales@medguard.ie)

Fax: 01 969 5009

Website: [www.medguard.ie](http://www.medguard.ie)

Unit 6/7, Block 4, Ashbourne Business Park, Ashbourne, Co. Meath, A84 VK52