

## INTRODUCTION

This walking stick can be adjusted in height. Made of aluminium which is light and resistant. The tip is covered with non-slip rubber.

Adjustable Height

COD. 27780 (from 71 to 94 cm)

COD. 27781 (from 67 to 90 cm)

COD. 27784 (from 77 to 100 cm): the length must be adjusted according to the height of the user.

## SAFETY RULES

Always contact your specialist dealer to adjust the device and to learn how to use it appropriately.

Do not store outdoors, do not expose it to prolonged periods of sunlight. Before using it check that all connections are tightened correctly.

Always move slowly, taking small steps to ensure that the weight distribution is even and that the position is stable and safe.

Do not use on slippery surfaces (wet floors, snow, ice) or on stairs.

Do not use for purposes other than those for which this product has been designed.

Always comply with the weight limits indicated on the label or in the instructions.

Use only original spare parts for repairs. Only qualified personnel are authorised to repair this item. Should you not use the device for long periods of time, have it checked by your specialist dealer.

Use caution when using on unstable and uneven ground. Check the wear of the non-slip rubber and replace it if necessary.

## INSTALLATION

The commissioning of this product should only be carried out by qualified personnel, incorrect adjustment could result in an accident.

Height adjustment: loosen the clamping ring. Press the button and slide the tube up to the desired adjustment hole.

Return the button to the starting position. Screw in the clamping ring.









To fold it, unhook and loosen the pipes.

## MAINTENANCE

The product must be cleaned regularly with a clean and dry cloth. The handle must be cleaned with a neutral detergent (do not use corrosive products). The ends must be checked and changed regularly in case of wear.

## GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

<b>REF</b>	IT Codice prodotto GB Product code		IT Leggere le istruzioni per l'uso GB Consult instructions for use		IT Attenzione: Leggere e seguire attentamente le istruzioni (avvertenze) per l'uso GB Caution: read instructions (warnings) carefully
<b>LOT</b>	Numero di lotto GB Lot number		IT Conservare in luogo fresco ed asciutto GB Keep in a cool, dry place		IT Rappresentante autorizzato nella Comunità europea GB Authorized representative in the European community
<b>MD</b>	IT Dispositivo medico GB Medical Device		IT Conservare al riparo dalla luce solare GB Keep away from sunlight		IT Dispositivo medico conforme al regolamento (UE) 2017/745 GB Medical Device compliant with Regulation (EU) 2017/745
	IT Fabbricante GB Manufacturer		IT Data di fabbricazione GB Date of manufacture		

## REF FS920L (27780) - FS939L (27781) - FS938L(12) (27784)



### Foshan Dongfang Medical Equipment Manufacturer (LTD.)

5 Xingfu Road, Zone A, Shishan Industrial Park,  
Nanhai District, Foshan City, China, 528000  
Made in China



### MedNet EC-Rep GmbH

Borkstrasse 10. 48163 Muenster.  
Germany

Medguard Professional Healthcare

Tel: +353 (0) 1 835 2411

Email: sales@medguard.ie

Fax: 01 969 5009

Website: www.medguard.ie

Unit 6/7, Block 4, Ashbourne Business Park, Ashbourne, Co. Meath, A84 VK52

