ratt rtarige sete								Outiliieu
Cutimed® Sorbact®	Description	Wound pad	Size	Order Code	Per box	When to use	Method of use	Frequency of change
Dressing pads	A Sorbact technology coated		7 x 9 cm	72161-01	1 x 5		Apply the group side of the drossing to the	
	swab with absorbent pad		7 x 9 cm	72161-00	1 x 40	Suitable for all types of contaminated, colonised or infected wounds with low to	Apply the green side of the dressing to the wound and retain with a bandage, tape or film dressing.	Can be left in place for up to seven
			10 x 10 cm	72162-01	1 x 5			days. Please use clinical judgement
			10 x 10 cm	72162-00	1 x 40	moderate exudate levels.	Cannot be cut.	depending on the wound condition.*
			10 x 20 cm	72163-00	1 x 20		Carmot be ede.	
Ribbon gauzes	Sorbact technology coated		2 x 50 cm	72166-00	1 x 20	All types of deep or cavity wounds showing signs of infection. Also suitable for fungal	Fill the wound with the dressing or place around digits or in skin folds if fungal	
	cotton gauze		5 x 200 cm	72167-00	1 x 10			Can be left in place for up to seven
						infection in the groin, in skin folds (e.g. under		days. Please use clinical judgement
						the breast) or between the digits.	infections are a problem.	depending on the wound condition.*
Swabs	Acetate swab coated with		4 x 6 cm	72164-01	1 x 5	Primary dressing for contaminated, colonised	Primary dressing for either deep or	
	Sorbact technology		4 x 6 cm	72164-00	1 x 40	or infected superficial or deep wounds. Also	superficial wounds. Swab dressings are sold	Can be left in place for up to seven
			7 x 9 cm	72165-01	1 x 5	suitable for fungal infection in the groin, in skin	as folded but are primarily used unfolded	days. Please use clinical judgement
			7 x 9 cm	72165-00	1 x 40	folds (e.g. under the breast) or	as an effective antimicrobial wound contact	depending on the wound condition.*
						between digits.	layer. Secondary dressing required.	
Round Swabs	Sorbact technology coated		3 cm	72168-00	1 x 70	Use for dressing small, deep wounds.	Retain with a secondary dressing.	Can be left in place for up to seven
	acetate fabric				(14 x 5)			days. Please use clinical judgement
					,			depending on the wound condition.*
Sorbact® Gel	A Sorbact technology swab		7.5 x 7.5 cm	72611-00	1 x 10	Can be used for all types of dry, sloughy or low exuding wounds that are contaminated, colonised or infected.	Apply directly to the wound. Cover with an appropriate secondary dressing that does not absorb the hydrogel.	Recommended dressing change
	coated with amorphous		7.5 x 15 cm	72611-01	1 x 10			interval is one - three days depending on the wound exudate levels.*
	hydrogel							
Sorbact® Hydroactive	A Sorbact technology coated	(5 x 6.5 cm)	7 x 8.5 cm	72646-03	1 x 10	Cotornsed of infrected.	absorb the flydroget.	on the would extudite levels.
	swab combined with an	(10 x 10 cm)	14 x 14 cm	72646-04	1 x 10			
	absorbent hydropolmer gel	(15 x 15 cm)	19 x 19 cm	72646-05	1 x 10	Can be used for low to moderately exuding	Use a bandage, tape or film to secure	The dressings should be changed as often as dictated by the wound
	sheet	(10 x 20 cm)	14 x 24 cm	72646-07	1 x 10			
		(20 x 20 cm)	24 x 24 cm	72646-06	1 x 5	wounds that are contaminated, colonised or	Cutimed® Sorbact® Hydroactive dressings.	condition or between one and four
	About the decoders 25	(5 (5 5 ))	7. 0.5	70022.00	4 40	infected. Also supports autolytic debridement and helps to maintain a moist wound environment.	Cutimed® Sorbact® Hydroactive B has an atraumatic adhesive border - no additional fixation required.	days. Change when the dressing is saturated (becomes cloudy or opaque).*
Sorbact® Hydroactive B	Absorbent hydrogel core with	(5 x 6.5 cm)	7 x 8.5 cm	79933-00	1 x 10			
Siltec® Sorbact®	sorbact technology for low to	(10 x 10 cm)	14 x 14 cm	79933-02 79933-03	1 x 10 1 x 10			
	moderately exuding wounds; adhesive border	(15 x 15 cm) (10 x 20 cm)	19 x 19 cm 14 x 24 cm	79933-04	1 x 10			
	adilesive boldei							
		(4 x 4 cm)	7.5 x 7.5 cm	73251-00	1 x 10			
		(6 x 6 cm)	10 x 10 cm	73251-16	1 x 10			
		(5 x 16.5 cm)	10 x 22.5 cm	73251-14	1 x 10			The dressing should be changed
		(8.2 x 8.2 cm) (10 x 10 cm)	12.5 x 12.5 cm 15 x 15 cm	73251-01 73251-02	1 x 10 1 x 10	For moderate to highly exuding, contaminated,		as often as dictated by the wound
	Sorbact technology with silicone	(12.3 x 12.3 cm)	17.5 x 17.5 cm	73251-02	1 x 10	colonised or infected wounds. Can also be used	Apply the pad to the wound. The dressings	condition or when the dressing is
	adhesive border	(16.5 x 16.5 cm)	22.5 x 22.5 cm	73251-03	1 x 5	as a secondary dressing for infected deep or	have a silicone adhesive border for retention.	saturated. Usually between one and
Silter® Seebast® (eval)						cavity wounds.		three days.*
Siltec® Sorbact® (oval)		(3 x 6 cm)	7 x 10 cm	73251-18	1 x 10			
Siltec® Sorbact® (sacrum)			17.5 x 17.5 cm	73251-05	1 x 5			
			23 x 23 cm	73251-06	1 x 5			
Sorbion® Sorbact®	Sorbact technology super	(8 x 9 cm)	10 x 10 cm	72698-00	1 x 10			- I
	absorbent dressing for highly	(7.7 x 17.8 cm)	10 x 20 cm	72698-01	1 x 10	For infected wounds or those at risk of infection	Apply ensuring green side of dressing has direct contact with wound bed, use appropriate fixation.	Dependent on condition of wound and clinical judgement. If conditions allow, may stay in place of up to seven days.
	exuding wounds	(17.8 x 17.8 cm)	20 x 20 cm	72698-02	1 x 10	with high to very high levels of exudate.		
		(17.5 x 27.5 cm)	20 x 30 cm	72698-03	1 x 10			
Leukomed® Sorbact®	Sorbact technology and post-	(2.8 x 3.8 cm)	5 x 7 cm	76199-00	1 x 20			
	operative dressing with a	(4 x 6.5 cm)	8 x 10 cm	76199-01	1 x 20			
	waterproof film	(4 x 11 cm)	8 x 15 cm	76199-02	1 x 20	For surgical incisions, post operative dehisced wounds, lacerations, cuts and abrasions with low to moderate exudate levels.	Apply with green pad in direct contact with wound, smooth film securely to the skin and carefully remove application film using the blue handling strip.  Dependent on wound condition i.e. exudate level and presence of infection. Dressing should be changed at least twice a week.	
		(5 x 15 cm)	10 x 20 cm	76199-03	1 x 20			•
		(5 x 20 cm)	10 x 25 cm	76199-04	1 x 20			•
		(5 x 25 cm)	10 x 30 cm	76199-05	1 x 20			
		(5 x 30 cm)	10 x 35 cm	76199-06	1 x 20			at least twice a week.
		(2.8 x 3.8 cm)	5 x 7.2 cm	76199-07	1 x 3	Without union a description of the Control of the C	Constitution of the state of th	than the constant delice
		(4 × 6 E cm)"	9 v 10 cm"	76100.00	1 v 2	^wnen using a dressing this may need to be changed r	more frequently for infected wounds, it is important to moni	tor the wound daily.

(4 x 6.5 cm)"

8 x 10 cm"

76199-08

1 x 3