

# Oxivir Excel Foam

## Cleaner & disinfectant for non-invasive medical devices and nonporous hard surfaces

### Description

Diversey Oxivir Excel combined cleaner & disinfectant, powered by AHP Technology, provides you with the best alternative to your traditional disinfectants by delivering a full virucidal efficacy in just 30 seconds in dirty conditions, while being very gentle to surfaces and with the lowest risk to your staff and customer. The active ingredient, hydrogen peroxide, breaks down into environmentally-friendly water and oxygen just minutes after use leaving no residues that might be harmful or change the appearance of the surface.

### Key properties

- Full virucidal claim in just 30 sec in dirty condition
- Fast kill claims for a broad spectrum of viruses, bacteria and yeasticidal in short contact times
- Fragrance free
- No rinsing required
- Safer for users as no CLP hazard classifications or symbols hence requiring no PPE
- Also safe for the environment, as the active ingredient breaks down into water and oxygen leaving no residues on the surface or pushing any harmful chemicals into the environmental waste stream.
- The short contact time, safe user profile and reduced PPE, improves labour efficiency by 35%

### Benefits

- Oxivir Excel has broad spectrum efficacy, providing the highest levels of protection against viruses, bacteria & yeast & has been extensively tested against European norms including EN14476, EN16615, EN13697, EN13727
- Oxivir Excel has fully virucidal efficacy with claims against all viruses including enveloped and non-enveloped viruses such as Coronavirus, VAccinia virus, Influenza, HIV, Poliovirus, Adenovirus, and Murine norovirus in 30 Seconds in dirty conditions.
- Short contact times keeps the surface wet for the required time, ensuring disinfectant efficacy
- Oxivir Excel is not only an excellent disinfectant but also an powerful cleaner, including descaling and lime limitation which removes the need for a pre-cleaning step
- Its excellent user safety profile reduces worker risk, with no CLP hazard warnings or classifications so does not require PPE & it doesn't contain respiratory irritants for improved air quality
- AHP formulation is gentle on surfaces extending equipment life as there are no harsh chemicals which leave residues on the surface
- AHP technology is also the most sustainable technology in the market as the active ingredient breaks down into water and oxygen so there are no harmful or harsh



The panel contains the following elements from top to bottom:

- A circular icon with a virus-like structure in the center. The text inside the circle reads "30 Seconds Fully Virucidal". Around the perimeter of the circle, the following viruses are listed: Poliovirus, Adenovirus, Norovirus, and Coronavirus.
- The CE mark followed by the number 0044.
- The AHP logo, which consists of the letters "AHP" in a bold, italicized font with a red dot below the "P".
- A grid icon with a magnifying glass over a surface, showing various shapes representing contaminants or residues.

*\*This dosage is according to optimal conditions, recommendations may vary, please consult with your Diversey representative for directions.*



# Oxivir Excel Foam

chemicals entering the waste stream

- Oxivir Excel is suitable for use in all general areas where there are hard surfaces in Hospitality, Education, Long Term Care, Retail, BSC sectors as well as high risk areas in Healthcare, including OR, ICU, NICU and patient rooms.
- Does not bind to cleaning tools like other actives such as quats, so disinfection is always optimal

## Use instructions

1. Remove gross soils.
2. Apply to surfaces by either spraying the cloth/ paper towel and wiping surfaces or applying directly to the surface by using a trigger sprayer.
3. After the contact time\*, either wipe dry or allow to air dry.

## Technical data

Appearance: Clear, colorless liquid

Odour: Neutral

pH value (in use): 1.95

Relative density (20°C): 1.000

Shelf life: Shelf life: 720 days

*The above data is typical of normal production and should not be taken as a specification.*

## Safe handling and storage information

Full guidance on the handling and disposal of this product is provided in a separate Safety Data Sheet; [sds.diversey.com](https://sds.diversey.com). Store in original closed containers away from extremes of temperature.

## Product compatibility

**Important:** Do not mix with other products. Do not use warm water. Do not use on water- or acid-sensitive surfaces.

Under recommended conditions of use, Diversey Oxivir Excel is suitable for use on most common hard surfaces (do not use on marble & brass)

## Microbiological data

### Environmental information

The surfactants used in this product are biodegradable in compliance with the requirement of EU Detergent Regulation, EC 648/2004.

## Available pack sizes

Oxivir Excel Foam is available in 6x0.75L Ready to Use, 2x5L Conc & Ready to Use Wipes (See separate PIS sheets for the conc and wipes) AHP® and Design and Oxivir® are trademarks of Diversey, Inc.



# Oxivir Excel Foam

Oxivir Excel Foam				
	EN test	Organims		750ml RTU
Bactericidal	EN 1276: 2010 FIDI	E.coli, S. aureus, P. aeruginosa, E. hirae	Dirty	1 Min
	EN 13727:2015 MED	S. aureus, P. aeruginosa, E. hirae	Dirty	30 Sec
	EN 13697: 2015 FIDI	E.coli, S. aureus, P. aeruginosa, E. hirae	Dirty	5 Min
	EN 16615: 2015 MED	S. aureus, P. aeruginosa, E. hirae	Dirty	1 Min
Virucidal	EN 14476: 2013 MED	Adenovirus /Poliovirus	Dirty	30 Sec
	EN 14476: 2013 MED	Murine Norovirus	Dirty	30 Sec
Yeasticidal	EN 1650, EN 13624, EN 13697	C. albicans	Dirty	5 Min
	EN 16615: 2015 MED	C. albicans	Dirty	1 Min
Fungicidal	EN 1650, EN 13624, EN 13697, EN 16615	Aspergillus Niger	Dirty	15 Min

Medguard Professional Healthcare

Tel: 01 835 2378

Email: [orders@medguard.ie](mailto:orders@medguard.ie)

Fax: 01 969 5009

Web: [www.medguard.ie](http://www.medguard.ie)

Unit 6/7, Block 4, Ashbourne Business Park, Ashbourne, Co. Meath, Ireland