

Emergency Smart Bags



Figure 1: Code: 27150



Figure 2: Code: 27151



Figure 3: Code 27153

FEATURES



GIMA bags and backpacks are designed to make first aid interventions more efficient. They assure safe transportation and the instruments, medications, products or medical and first aid devices are stored in practical compartments.

GIMA bags and backpacks are made with very sturdy, anti-scratch and water-proof materials, with excellent closures and finishing, to avoid jamming or breaks during ordinary use. The bags and backpacks are supplied empty.

PRESCRIPTIONS

The product can be used only by qualified people, aware of the hygienic limitations due to the use of the product in places where there could be polluting or bacteriologically infected elements. GIMA cannot be held responsible for damages caused to the content or that the contents of the bag may cause to the bags or backpacks.

The product, or any of its parts, cannot be used for purposes other than those it was designed for.



Do not leave the product unattended, in the reach of children or unqualified personnel.

Do not use the product if it is damaged, contact dealer.

Avoid any temporary repair.

During use be careful not to damage the zippers.

When the zippers are being opened and closed verify that they do not get jammed into other materials and always open or close them completely.

If any liquids or powders should leak out of bottles, clean and disinfect the bag or backpack, because they could damage or contaminate the bag itself or the products inside.



Do not fill up too much. The weight and shape of the content must fit the type of bag or backpack. Always verify that the weight of the content does not exceed the weight indications of the bag model or the possibility of the user.

(according to Italian laws an operator can lift maximum 30 kg).

UNPACKING



The elements of the packaging (paper, cellophane, metal staples, adhesive tape, etc.) may cut and/or hurt if not handled with care. Remove them with appropriate tools and do not leave with unauthorized people. The same goes for the tools used to open the packaging (scissors, knives, etc.).

The first operation to do after opening the packaging is to control all the parts of the product; verify that all the components needed are in place and verify their conditions: movement of zippers, fastening of clips, Velcro or strips, resistance of handles.


MAINTENANCE

To clean the bags and backpacks we suggest washing with hot water and appropriate detergents. The bags can also be washed in a washing machine. Do not spin the product. After washing we suggest disinfecting the product in a diluted solution or spray. Before putting the product in the washing machine, remove all the padding and rigid parts.

The bags and backpacks cannot be put in an autoclave unless specifically indicated on packaging.

We suggest using a specific, inert and bacteriologically pure lubricant to assure the proper operation of the zippers: Vaseline.

Periodically control the integrity of the product, verifying the closure of the zipper, shoulder straps, Velcro, handles and belts. If they are damaged or too worn do not use the product and replace or repair immediately.

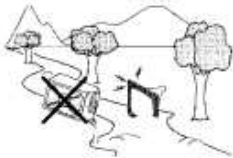

Comply with the laws in the country of use to properly dispose of the product.